

**COSMEDENT®**

*Spreading Smiles™*



## COLOR CHANGE VENEER


Dr. Buddy Mopper




Color Change Veneer



Notice the Tetracycline stain

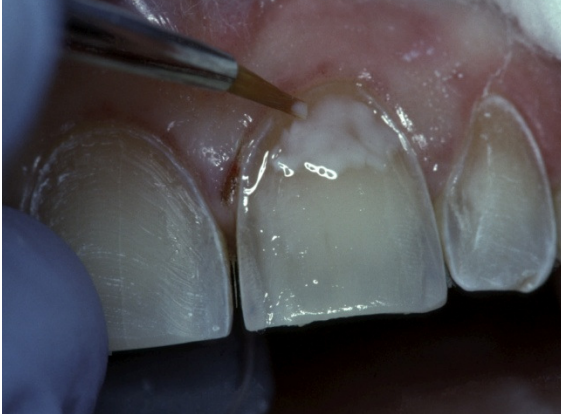


After etching and placement of Cosmedent Complete bonding agent, pink opaque is placed using Cosmedent's #1 brush

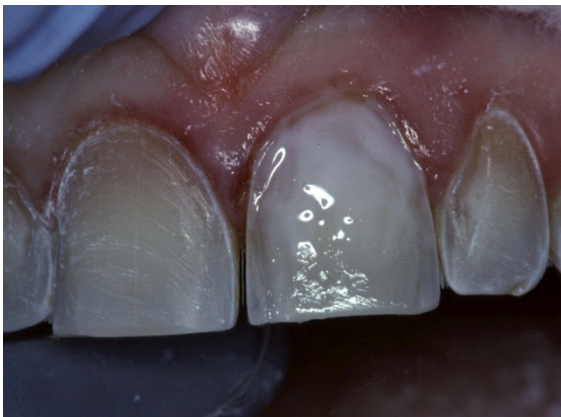

Pink opaque cured for 20 seconds




Application of Renamel  
Microhybrid

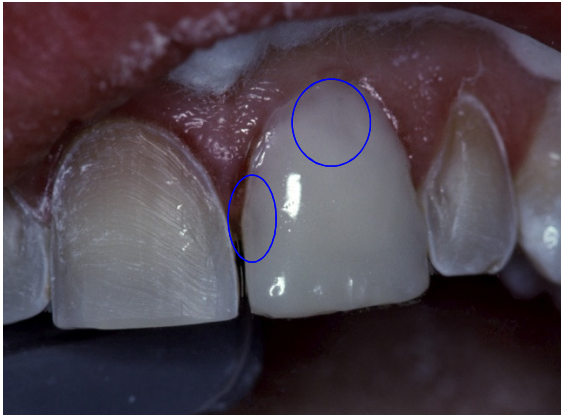

After curing Microhybrid for 20  
seconds, notice the slight shine  
through



Use Creative Color Opaque to block  
out shine through areas, and  
increase value.

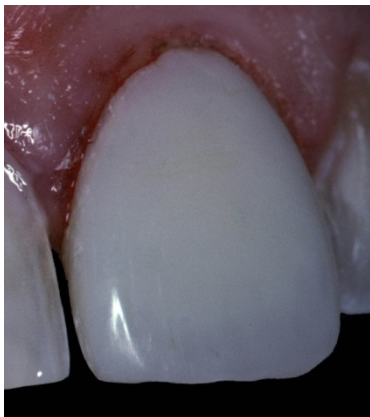

Microfill layer placed over surface,  
of tooth, cured 60 seconds and  
Polished



Before



After