

**COSMEDENT®**

*Spreading Smiles™*



## GINGIVAL CROWN MARGIN REPAIR

Dr. Dennis Hartlieb



Gingival crown margin repair



Application of GINGA-Guard™ light cured rubber dam. Notice the long bevel on the porcelain



Cosmedent Clear Hydrofluoric Acid on porcelain for 3 1/2 minutes. Rinse away with copious amounts of water



Notice completely etched surface of porcelain



Remove GINGA-Guard, and etch dentin using 37% phosphoric acid, rinse with copious amounts of water



Application of bonding agent after the placement of silanator on porcelain only



Application of Creative Color Pink Opaque applied to dark surface only, cure for 20 seconds



Application of Renamel Microhybrid



Application of Renamel Microfill same shade as Renamel Microhybrid, cure for 60 seconds



Now that's invisible!